



Zest Dental Solutions  
www.zest.com

94%  
overall  
rating

## Description

**Cem EZ®** is a dual-cured, universal resin cement that is delivered in automix syringes with ready-to-use mixing tips. It is formulated to offer excellent bond strength to enamel, dentin and indirect restorative materials. **Cem EZ** can be used with a self-etch, total-etch, or selective-etch techniques. Featuring the proprietary *Intellitek Technology*, this universal cement provides long-term color stability and compatibility with *Prelude One* as well as a range of other adhesives. It can be tack cured in two seconds for easy clean-up. **Cem EZ** is available as a starter kit containing three 5 g syringes (one each translucent, white opaque and warm shades), 24 automix tips, 5 mL *Sure Etch Gel*, 12 etch tips, 5 mL *Prelude One*, 40 application brushes, and a reusable mixing well. Refill kits are also available containing two syringes of one shade together with 17 application brushes. Separate automix tips are available in packs of 24.

## Unique Features

- Inclusive of the proprietary Intellitek Technology for long-term color stability and adhesive compatibility
- Protection against microleakage
- Esthetic cement with high bond strengths
- Automix delivery system

“EASY AUTOMIX  
AND DISPENSING,  
AND GOOD  
VISCOSITY.”

## Consultants' Comments

- “Great product and an esthetic cement for anterior teeth - good shade options.”
- “Excellent viscosity for endo post cementation.”
- “Clean-up was a breeze after tack-curing this cement.”
- “I liked its compatibility with a variety of adhesives.”
- “I never saw a finish line, **Cem EZ** has a chameleon effect and disappears.”
- “This is a very efficient product.”
- “The self-cure time seemed to be longer than other cements.”
- “No post-operative sensitivity.”
- “Cementation of composite seemed to have a lot of steps.”
- “The working and setting times are very short. This is good for single crowns but would be an issue for multiple restorations or a long multi-unit bridge.”



## Indications

- Permanent cementation of crowns, bridges, inlays and onlays fabricated from zirconia, ceramics, metals and composites
- Cementation of all types of endodontic posts

## Clinical Tips


- A quick two-second tack cure makes clean-up easy. Do not over-cure before clean-up.
- The laminated instruction card is very useful.

## Evaluation Highlights


**CemEZ** was evaluated by 27 consultants, and used 429 times in total.

- Easy to automix and dispense
- Use with self-etch, selective-etch and total-etch techniques
- Flexible and multi-purpose
- Lack of post-operative sensitivity
- Esthetic results

### Key Features:

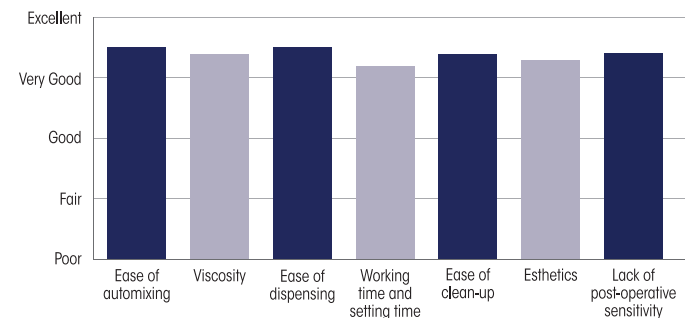

### Compared to Competitive Products:

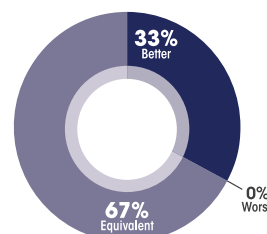

### Percentage of Consultants Who Would:

